

Discover **Activa**® GRADUATED COMPRESSION LEGWEAR



Ultra-Sheer Lite Support - Preventive Support

Light Compression: 9-12 mm Hg
Helps prevent and relieve tired, aching legs; mild ankle and foot swelling; and mild spider and varicose veins.

- H11 PANTYHOSE - Control Top Suntan, Black
- H12 THIGH HIGH - Lace Top Suntan, Black
- H13 KNEE HIGH Suntan, Black



Sheer Therapy® - Preventive Support

Light Compression: 15-20 mm Hg
Helps prevent and relieve leg fatigue; ankle and foot swelling; and mild varicosities without edema.

- H21 PANTYHOSE White, Nude, Smoke, Black
- H22 THIGH HIGH - Lace Top Nude, Smoke, Black
- H23 KNEE HIGH White, Nude, Black
- H29 MATERNITY PANTYHOSE Nude
- H202 THIGH HIGH - Open Toe Nude
- H204 KNEE HIGH - Open Toe Nude



Soft Fit Graduated Therapy with Microfiber

Moderate Compression: 20-30 mm Hg
Indicated for relief of chronic leg fatigue and heaviness; ankle and foot swelling; mild varicosities without significant edema; and Post Sclerotherapy.

- H37 PANTYHOSE Barely Beige, Ivory, Black
- H38 THIGH HIGH - Lace Top Barely Beige, Ivory, Black
- H39 THIGH HIGH - Uni-Band Top Barely Beige, Black
- H304 KNEE HIGH - Open Toe Beige
- H30 KNEE HIGH - Closed Toe Barely Beige, Ivory, Black



Surgical Weight Graduated Therapy Heavy Compression: 30-40 mm Hg

Indicated for relief of chronic leg fatigue and heaviness; ankle and foot swelling; chronic venous insufficiencies; Post Sclerotherapy; Post Surgical Stripping; and Prophylaxis of Thrombosis.

- H42 THIGH HIGH Beige
- H43 KNEE HIGH - Closed Toe Beige
- H44 KNEE HIGH - Open Toe Beige



Men's & Women's Trouser Dress Socks Light Support: 15-20 mm Hg Graduated Compression

Helps prevent and relieve leg fatigue; ankle and foot swelling; and mild varicosities. Ideal for individuals who sit or stand for long periods of time and for frequent air travelers.



SOLID COLOR:

- H25 MEN'S Black, Navy, Brown, White
- H26 WOMEN'S Black, Navy, Tan, White

PATTERNED STYLE:

- H24 MEN'S Black, Navy
- H27 WOMEN'S - Diamond Black, Navy, Tan
- H28 WOMEN'S - Cross-Hatch Black, Navy, Tan



Men's Dress Socks Firm Support: 20-30 mm Hg Graduated Compression

Indicated for relief of chronic leg fatigue and heaviness; ankle and foot swelling; and mild venous insufficiencies without significant edema.

- H35 MEN'S DRESS SOCKS Black, Navy, Brown, Tan, White
- H34 MICROFIBER PIN STRIPE STYLE Black, Gray, Tan



CoolMax® Athletic Socks for Men and Women Firm Support: 20-30 mm Hg Graduated Compression

Indicated for relief of chronic leg fatigue and heaviness; ankle and foot swelling; and mild venous insufficiencies without significant edema. Ideal for athletic or work wear.

- H313 CREW LENGTH White
- H312 OVER-THE-CALF LENGTH White



Anti-Embolism Stockings 18 mm Hg Graduated Compression



Indicated for Thrombo Embolic Deterrent; non-ambulatory patients; pre- and post-operative procedures; and mild varicosities without significant edema.

- THIGH HIGH - Inspection Toe White
- THIGH HIGH - Open Toe Beige
- THIGH HIGH - Closed Toe White, Beige
- KNEE HIGH - Open Toe Beige
- KNEE HIGH - Closed Toe White, Beige

APPLICATION AND CARE INSTRUCTIONS

Apply Activa® Legwear in the morning upon rising before any possible swelling. For easier application, feet and legs should be completely dry.

1. Gather one stocking or sock from top to toe and draw over foot.
2. Position heel in stocking, place foot on floor and slowly start pulling the stocking up the leg. Smooth out wrinkles while pulling stocking up. Adjust toe for freedom of movement. For pantyhose products, continue to pull product evenly to waist and hips and adjust for comfort over groin and buttocks.
3. To remove, slowly start from top pulling downward turning the stocking inside out.

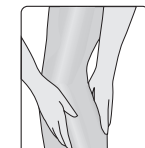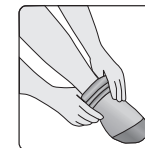

IMPORTANT: Rough hands, fingernails and jewelry can run and snag stockings. Do not cut any loose ends.

CARE INSTRUCTIONS:

FOR HOSIERY: To extend product life, hand wash in warm water with mild soap.

FOR COMPRESSIVE SOCKS: To extend product life, hand or machine wash in warm water with mild soap.

FOR BOTH: Do not bleach. Blot to remove excess water. Air dry away from heat. Do not machine wash or dry.

If hosiery is extremely stretched out after wear, washing in slightly warmer water may help return them to original shape.

For more information, visit www.ActivaHosiery.com



Newly Released

Complements - Sheer Support

Moderate Compression: 20-30 mm Hg

Sheer and fashionable graduated compression relieves tired, aching legs, foot and ankle swelling and discomfort associated with varicose veins. Indicated following vein surgery or sclerotherapy. Doctor recommended to improve circulation and reduce swelling.

- H71 PANTYHOSE (True Beige, Classic Bronze, Cocoa)
- H72 THIGH HIGH (True Beige, Classic Bronze, Cocoa)
- H73 KNEE HIGH - Open Toe
- H74 KNEE HIGH - Closed Toe

} (True Beige, Classic Bronze, Cocoa)

